

Doctor Recommended

**ThinZ[®] Extra Sensitive
MetaboSTRIPS[™]**

The Diet Diagnostic[®]

DIRECTIONS: 1) Remove one (1) test strip from the special humidity resistant container and immediately re-close container. Do not touch *test pad* area of strip. 2) Dip the *test pad* area of the strip into a fresh urine sample or pass it through urine stream for 1 or 2 seconds only. Tap excess urine off the *test pad*. 3) Wait 1 to 2 minutes after exposure to urine, then immediately compare the *test pad* color to the color chart on the container label. If *test pad* color falls within, or is darker than **detected** range it indicates that fat is being metabolized. Remember, read results within 2 minutes after exposure to urine, as results after 3 minutes may not be reliable. Test once daily, *before breakfast*, starting 2 to 4 days after beginning a sensible reduced calorie diet and exercise program. Continue monitoring until desired goals are obtained and thereafter as desired. NOTE: It is not uncommon for *test pad* results to show only a slight color change, depending on your degree of calorie reduction and increased exercise. Consuming certain foods or drugs which discolor normal urine may cause results to vary. See package insert for additional important information.

REAGENT: Buffered Sodium Nitroprusside

CAUTION: THINZ[®] MetaboSTRIPS[™] Test Strips are intended for use by normally healthy adult persons only, as an addition to a sensible *weight loss or weight control* program consisting of a reduced calorie diet and regular moderate physical exercise. It is always wise and recommended to consult a doctor before beginning any weight loss program and advise him/her of the methods you intend to use to achieve and monitor your fat metabolism...and to periodically keep him/her informed of your progress and results. This product is not intended to diagnose any disease or nutritional deficiency.

For best results, THINZ MetaboSTRIPS Test Strips should be used within 6 months of first opening the sealed container.

KEEP OUT OF REACH OF CHILDREN
KEEP CONTAINER TIGHTLY CLOSED WHEN NOT IN USE. STORE
AWAY FROM DIRECT SUNLIGHT, EXCESSIVE HEAT, COLD AND HUMIDITY.

Date: 0306

Revision # 99-0R, 0605